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#### **GB** Instructions for use

**Type of application:** As a medical device: Ready-to-use alcoholic wet wipes for the prophylactic surface disinfection of alcohol-resistant medical devices. Not suitable for the final (terminal) disinfection of invasive medical devices.

As a biocidal product: Ready-to-use alcoholic wet wipes for the disinfection of small alcohol-resistant areas.

**Notes on application:** Open the top of the resealable dispenser bag and remove a soaked wipe. Then immediately reseal the bag tightly. Carefully wipe the surfaces to be disinfected and allow the disinfectant to take effect. All surfaces to be disinfected must be completely wetted. The wipes can be used for a maximum of 28 days after the bag has been opened for the first time. Before opening the bag for the first time, the opening and expiry date must be documented manually on the sticker or in the field provided. Use only on alcohol-resistant materials (not suitable for acrylic glass). Suitable for the medical sector ((dental) practices, clinics, nursing homes, care and rescue services) and the food, industrial, household and public sectors.

**Application times (of the impregnation fluid):** Bacteria and Candida albicans as per EN 13727 and EN 13624 (dirty conditions): Conc. 1 min., TB (tuberculosis) inactivation (M. terrae) as per EN 14348 (dirty conditions): Conc. 1 min., Enveloped viruses (e.g. HBV, HIV, HCV) as per EN 14476 and DVV/2012 in a surface test (dirty conditions): Conc. 1 min., "Limited virucidal PLUS" (noro-, rota- and adenoviruses) (dirty conditions): Conc. 1 min.

**Application times of the impregnated wipes:** Bacteria and Candida albicans as per EN 16615, VAH under dirty conditions: 1 min.

**100g impregnating fluid contain:** 43.0 g ethanol, 8.0 g 1-propanol; 0.055 g didecyldimethylammonium chloride

**Contents:** 80 pcs. made of 100% PET, 25 cm x 25 cm, impregnated with Rapid Surface Disinfection Omnizid

**Safety Instructions:** H 228 Flammable solid. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Wear eye protection/ face protection. Store in a closed container. Only return the packaging to the recycling collection when completely empty. Dispose of any remaining filling material as per EAK 07 06 99.

(EC) 648/2004 (impregnation liquid): Contains: fragrances

##### **Notes on incidents:**

All serious incidents related to the device shall be reported by the user/patient to the manufacturer and to the competent authority of the Member State in which the user/patient is established.

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BAUA-Nr.: N-72848

UFI:1AFC-JN7C-H10P-K0H8

PRISMAN GmbH

Otto-Hahn-Ring 6-18

D-64653 Lorsch

Tel.: +49 (0) 6251 866980-0

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OMNIDENT Dental-  
Handelsgesellschaft mbH

Gutenbergring 5

D-63110 Rodgau Tel.: +49

(0) 6106 874-0