
# Omni-Etch

Ätzgel 37 % auf Phosphorsäurebasis  
*Phosphoric acid etching gel 37 %*  
*Etsgel op basis van fosforzuur 37 %*  
*Gel mordenzante a base di acido*  
*fosforico 37 %*

**Inhalt:** 20 x 1,4 ml (1,9g) Applikationsspritzen  
**Content:** 20 x 1,4 ml (1,9g) Etching Gel  
**Inhoud:** 20 x 1,4 ml (1,9g) Etsgel  
**Contenuto:** 20 x 1,4 ml (1,9g) Gel siringa

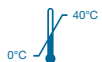REF 219261

Rev. 2405\_00/2024-05

PRISMAN GmbH  
 Otto-Hahn-Ring 6-18  
 D-64653 Lorsch  
 Tel.: +49 (0) 6251 866980-0

Gefahr / Danger  
 Gevaar / Pericolo

OMNIDENT Dental-  
 Handelsgesellschaft mbH  
 Gutenbergring 5  
 D-63110 Rodgau

## DE: Anwendung:

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## EN: Notes on application:

Phosphoric acid-based etching gel for the treatment of tooth enamel and dentin prior to dental treatments.

## Notes on application:

Etching gel is a 37% phosphoric acid etching gel with optimal thixotropic flow characteristics. It is used for etching tooth substance within the framework of the enamel etching technique and total-etch technique. According to current knowledge, the product has no side effects if the safety and application instructions are observed. Underfilling materials can be dissolved by the etching gel during longer exposure times.

- For dental use only!
- Do not use beyond the shelf-life date
- Close refill syringe immediately after use!

## Instruction for use:

The sealing cap is twisted off counterclockwise and an applicator cannula is attached.

## Preparation:

The areas of the tooth substance to be etched must be clearly delimited.

- 1.) The use of a rubber dam is recommended! Any contact with soft tissue (skin, oral mucosa, eyes etc.) is to be avoided. If etching gel gets into the soft tissue, immediately rinse the area with plenty of water and, if necessary, contact a doctor.
- 2.) Before preparation, clean the tooth using a rubber cup and fluoride- and fat-free polishing paste.
- 3.) Use water to properly remove cleaning residues, clean, and dry the surface of the tooth with oil-free compressed air.
- 4.) Prepare the cavity accordingly; bevel the melted edges to achieve permanent enamel junctions. Rinse the cavity thoroughly with water and dry with oil-free compressed air. Avoid overdrying!
- 5.) Protect dental pulp in deep cavities (e.g. cover with a thin layer of suitable underfilling material). The cavity should be dry and free of contamination, whereby the overdrying of the dentine is to be prevented.

## Etching process

- 6.) Apply etching gel to the enamel margins of the cavity and, if necessary, to the dentin.

Recommended application times:

- 15 - 30 sec. on tooth enamel
- 10 - 15 sec. on dentin
- 50 - 80 sec. on milk teeth and teeth with a high content of fluoride

- 7.) Rinse with water for at least 20 seconds, no etching

gel should be left.

- 8.) Dry with oil- and water-free compressed air. The etched area must not be touched or contaminated by saliva until the treatment is completed. However, should contamination occur after the etching and drying, the area must be etched again briefly (approx. 10 sec.).

The etched enamel should look chalky white! After etching, you can continue with the application of adhesives etc. in accordance with the manufacturer's instructions for use. Do not reuse application cannulas! Smaller quantities of etching gel diluted with water can be disposed of via the sewage system. Dispose of uncleaned packaging according to local regulations. Application is restricted to professional users!

## Safety Instructions:

H314 Causes severe skin burns and eye damage.

P280 Wear protective gloves and eye protection.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do. Continue rinsing. P310 Immediately call a POISON CENTER or doctor/physician. P301+330+331 IF SWALLOWED: Rinse mouth. DO NOT induce vomiting. P302+P352 IF ON SKIN: Wash with plenty of soap and water. P405 Store locked up. P501 Dispose of contents/ container in accordance with all local, regional, national and international regulations.

## Notes on incidents:

All serious incidents related to the device shall be reported by the user/patient to the manufacturer and to the competent authority of the State in which the user/patient is established.

## Ingredients:

Contains 37% ortho-phosphoric acid, pyrogenic silicon dioxide, humectants and dyes.

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